AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

LumboLoc is a medical device. It is an orthosis¹ for the stabilization and relief of the lumbar spine.

 **LumboLoc must be individually adjusted for the patient by a trained specialist². This is the only way to guarantee the full effect and ideal wearing comfort of the orthosis. If the product is not used properly, any claim for warranty will expire.**

Indications

- General pain in the lumbar spine (acute and chronic)
- Moderate degeneration / muscular insufficiency
- Facet syndrome

Risks of using this product

- The use and duration of use must be discussed with the physician who prescribed the LumboLoc. He will also inform you whether the LumboLoc can be used simultaneously with other products, and if so, which ones. Please note all the information you received with the LumboLoc as well as the instructions for use.
- The product may be damaged if it is not used correctly or is altered arbitrarily and, the warranty will not be valid.
- The product must be applied over a T-shirt or an undershirt.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- Correct usage/fitting is assumed. Any aids applied externally to the body may, if tightened excessively, lead to local pressure points or, rarely, constrict the underlying blood vessels or nerves.
- If you experience unusual changes connected to the use of the LumboLoc – e.g. an increase in discomfort – stop using it and contact a physician immediately.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

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
Side effects involving the body as a whole have not been reported to date. In the following conditions such aids should only be fitted and applied after consultation with your medical advisor:
1. Skin disorders / injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with

swelling, redness and excessive heat build-up.

2. Impaired sensation.
3. Severe restriction of the performance of heart and lungs (risk of a rise in blood pressure if the aid is worn during severe physical exertion).

Application instructions

Fitting LumboLoc

- Wrap the product around your hips so that the stays are positioned on your back .
- Place your fingers in the straps specially provided on the right and left hand fastenings and uniformly pull the support forwards .
- Press the left hand fastenings against your abdomen and then pull the right hand fastenings until the two halves can be secured .
- When securing the Velcro fastening withdraw your fingers from the left strap first, followed by the right strap .

Cleaning instructions

We recommend washing the LumboLoc separately by hand at 30 °C using a mild detergent. To prevent damage and to preserve their long-term adhesive strength, we recommend closing the Velcro fastenings before washing. Do not dry in direct heat (e.g. heater, sunlight, etc.).

This product has been tested by our own quality control system. However, should you have any complaints, please contact your local medical supply retailer.

Part of the body this product is used for

In accordance with the indications. See intended use.

Technical parameters

LumboLoc is made from knitted fabric with integrated dorsal corset stays.

Assembly and fitting instructions

LumboLoc is supplied ready for use.

Maintenance information

If handled and cared for correctly, the product is practically maintenance-free.

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. It is your first local point of contact before the manufacturer. Please use the LumboLoc in line with expert instructions and the enclosed instructions for use. Take note of the care instructions and do not make any alterations to the product. Only aids that function perfectly will help you. Failure to follow the instructions may also compromise the warranty.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyamide (PA), Polyurethane (PUR), Spring Steel Strip C75, coated, Elastodiene (ED), Viskose (CV), Cotton (CO), Thermoplastisches Polyurethan (TPU), Polyester (PES), Spandex (EL)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2020-08

¹ Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso.

² A specialist refers to any person who is authorized according to the state regulations for fitting and instruction in the use of orthoses which are relevant to you.

Instructions for the orthopedic technician

If necessary the stays can be adjusted.

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