AUSTRIA

Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0) 800 4430-130
F +43 (0) 800 4430-131
E info@bauerfeind.at

BENELUX

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read through these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Purpose

The MalleoTrain S is a medical device for the stabilization of the ankle.

Indications

- Ligament insufficiency
- Supination prevention
- Slight sprains

Risks of using this product

To date, there are no known side effects that impact the entire body. The product must be put on / used properly. Do not put the MalleoTrain S on too tightly or it could create localized pressure symptoms. In rare cases constriction of the blood vessels and nerves is possible.

 Contains natural rubber latex (elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

There are no known medically significant hypersensitivities. With the following clinical presentations you should only put on and wear this type of medical after consulting with your doctor:

- Skin problems / injuries to the part of the body where the support is worn, especially if there are signs of inflammation. Also, raised scars with swelling, redness and overheating
- Sensation / movement disorders in the legs / feet, e.g., associated with diabetes
- Lymphatic drainage problems, including undiagnosed soft tissue swelling in a part of the body other than where the support is worn.

Information about wearing the product

Do not use with other products, such as compression stockings, unless you discuss this in advance with your doctor. Do not let the product come in contact with substances containing fats or acids, ointments or lotions. If your symptoms worsen or you notice any unusual changes, please contact your doctor immediately.

The manufacturer is exempt from product liability if the product is used improperly. This product is most effective when the wearer is physically active.

Take the MalleoTrain S off if resting for long periods (e.g., sleeping, sitting for a long time). Do not make any changes to this product. If this restriction is not observed, the product's performance may be compromised so that product liability is excluded.

Pulling / putting on the MalleoTrain S

- 1 Pull on the support so that the start of the strap is located on the inside of the foot. The start of the strap defines the direction in which the strap needs to be wound around the foot. When fitting the strap, please make sure there are no creases.
- 2 Bring the strap under the sole of your foot toward the outside of your foot. By pulling the strap you can stabilize the foot and ankle comfortably.
- 3 Now bring the strap diagonally across the top of the foot and back around the ankle mortise.
- 4 5 Bring the strap diagonally forward again to the inside of the foot.
- 6 Pull the strap under the sole of your foot toward the outside of your foot and over the top of your foot.
- 7 Now secure the end of the strap by fastening its Velcro strip to the section of the strap which is underneath it.

Alternatively you can use the designs on the support to guide you. Blue dots on the titanium-colored knit fabric indicate where the strap is supposed to go. When wrapping the strap around your foot, follow the dots knitted into the fabric, starting with one dot, going to two dots and then to three dots.

Cleaning instructions

Please use mild detergent and a mesh bag. Never expose your product to direct heat / cold. Also observe the instructions on the sewn-in label on the upper edge of your product. Regular care guarantees optimum effectiveness of the product.

Close the Velcro fastener to prevent damage from occurring.

Notes on reuse

This product is intended for your treatment only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the ViscoPed S have not been observed.

Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authority. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

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