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UNITED ARAB EMIRATES

Bauerfeind Middle East FZ LLC
Dubai Healthcare City
Building 40, P.O. Box 50511
Dubai

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All instructions for use:
www.bauerfeind.com/downloads

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en english

Dear Customer,

thank you for choosing a Bauerfeind product that meets very high medical and quality standards.

You will find enclosed a kit comprising: individually milled blanks, outer soles and straps. Please check these blanks in particular to see if they comply with your specifications, and notify us of any potential discrepancies before you proceed with any further processing steps. The kit must still be modified and processed by you in accordance with the prescriber's / health insurance provider's specifications and in consideration of current industry knowledge before it becomes the final product. Legal stipulations also require that you explain the intended purpose and use to the consumer. Please enclose your instructions for use with the finished product when handing it over.

Intended purpose

Milled toe separators / sandals are products intended for prevention and can assist with the conservative treatment of foot problems.

Indications

- Flat foot / splay foot
- Skew foot
- High-arched foot
- depending on the structural design

Please note the following modification instructions:

- The product comprises the individually milled footbed, strap (toe separator) or shaft and midsole as well as outer sole.
- The footbed of the toe separator / sandal must be modified by trained specialists* according to the consumer's needs (shape, fit, material thickness) for a proper fit.
- Carry out a standing test, mark the position of the strap or shaft at the footbed and mark the point at which the toe separator (to strap) passes through on top of the footbed.
- Shorten the ends of the straps / shafts according to the customer's instructions and glue them into the cutouts on the bottom of the footbed using standard contact glue.
- Then glue the outer sole, midsole and footbed together using standard contact glue. Then smooth the contour with a final polish all around.
- Instruct the consumer how to use and clean the product.
- We accept no liability for any further processing and modification steps carried out by you.

Material content

Shaft / strap: Polyurethane (PUR), Polyester (PES), real leather

Footbed: Polyolefin

Midsole: Polyolefin

Sole: Styrene-butadiene rubber (SBR)

Storage

Do not expose the components of the set to direct UV light. Store them in a cool, dry place and keep them uncovered. Regular checks of the storage conditions are recommended.

Cleaning instructions

If needed, all surfaces can be cleaned with a mild water-soap solution (room temperature) using a sponge or cleaning cloth. Do not clean them in a large quantity of water. Any residual soap must be thoroughly rinsed off with water. The kit and its components are not machine-washable. The shoes are not suitable for use in water. Do not expose the shoes / the kit to direct heat or cold. Mild disinfectants suitable for human skin may be applied sparingly. This may lead to color changes.

Disposal

Dispose of the shoes / the kit with household waste in compliance with regional specifications.

Notes on reuse

The product is designed to provide care for an individual patient / individual foot orthoses wearer and must not be reused or used by any other person. The manufacturer accepts no liability in the event of improper use of the product. Foot orthoses are affected by various mechanical forces (body weight, tension, pressure, torsion, friction), with the result that their serviceable life depends on the load. Temperatures, stacking materials, the properties of the orthosis materials, shoes and environmental influences (e.g. perspiration) all play their part in reducing the functional life of the foot orthosis. After a foot orthosis has been

worn for a maximum of six months, the manufacturer recommends that the orthopedic functions of the orthosis are checked by a qualified specialist. The maximum wearing period of six months can be reduced on a case by case basis depending on the level of wear.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims.

Warranty may be limited or excluded if the instructions on how to handle and care for the GenuTrain have not been observed.

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